



XVCapture

INSTALLATION GUIDE

System Requirements:


- **2.0 GHz or higher processor (faster is always better, especially for servers)**
- **4GB of RAM (again, more is always better)**
- **500 GB Hard Drive**
- **CD-R/DVDR drive**
- **100/1000Mbps or higher Local Area Network (TCP/IP preferred)**

Pre-Installation


Your Implementation Coordinator will arrange a call with a member of our installation team to provide you with the necessary files for the software installation. After the meeting, you can follow the steps below and using the files provided to install Aptyeryx XVCapture on your devices.

Installation


1. Copy the provided **Aptyeryx** folder to the root of the **C:** drive on each computer that will need to be configured as an x-ray capture PC. You should end up with a **C:\Aptyeryx** file path. You can also copy this folder from an existing PC that is already up and running.

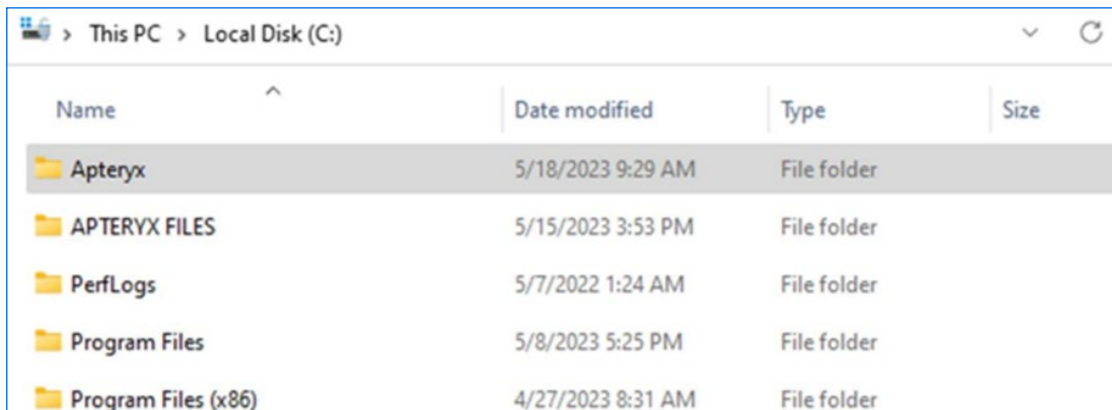


2. Open the **Aptyeryx** folder, then open the **XVCapture** folder.

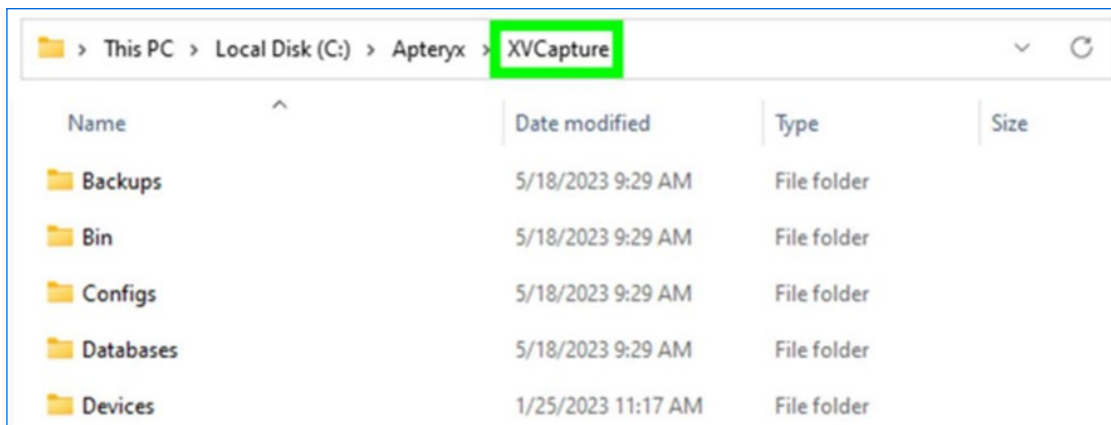


3. Scroll down to the bottom of the file list and right-click and run the **vc_redist.x86.exe** file as Administrator.

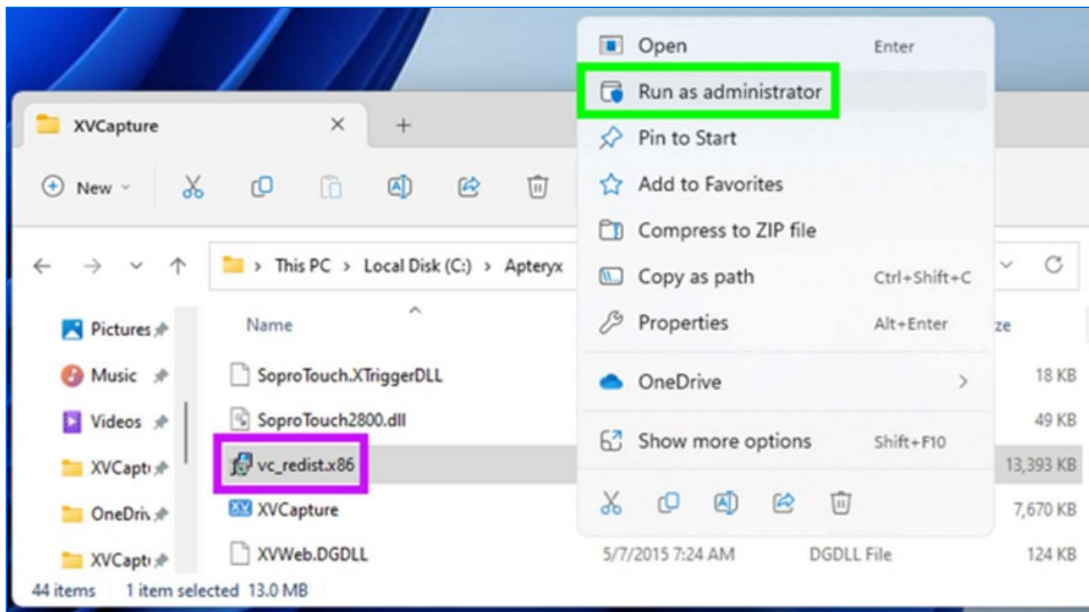


4. Right-click and run the **XVCapture.exe** application as Administrator.

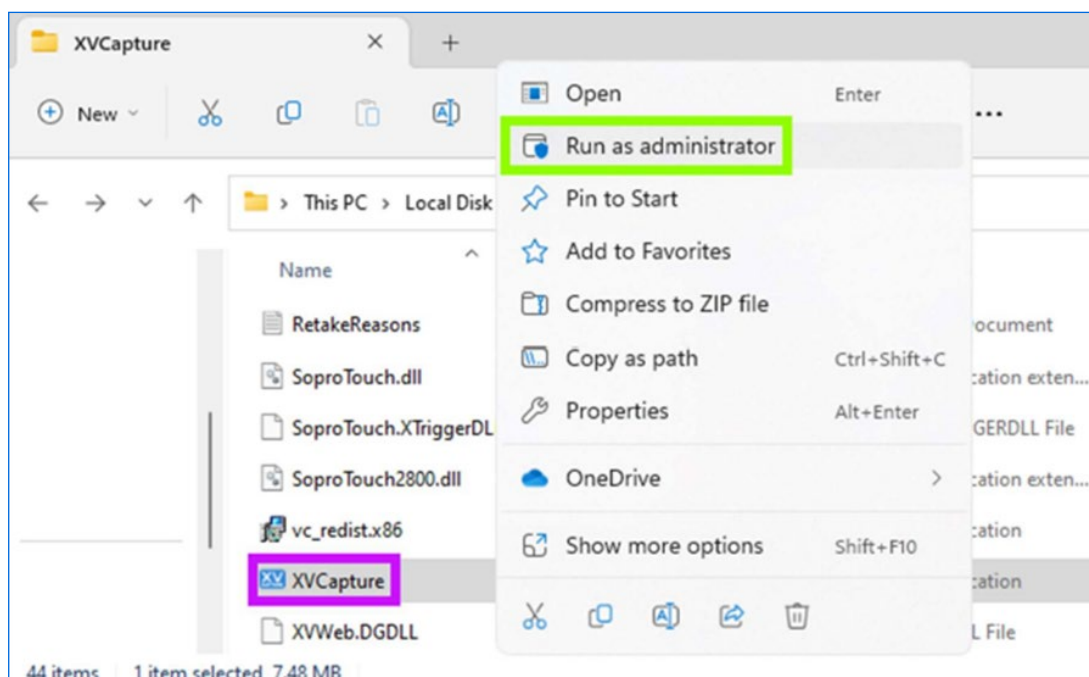


5. When the **Assign the Default Configuration** window appears click **Yes**, then on the **Setup Application Shortcuts?** window check the **Do not display this message again** checkbox, and finally click the **Yes, create the shortcuts** button. XVCapture will open, but you can close it back out after that.

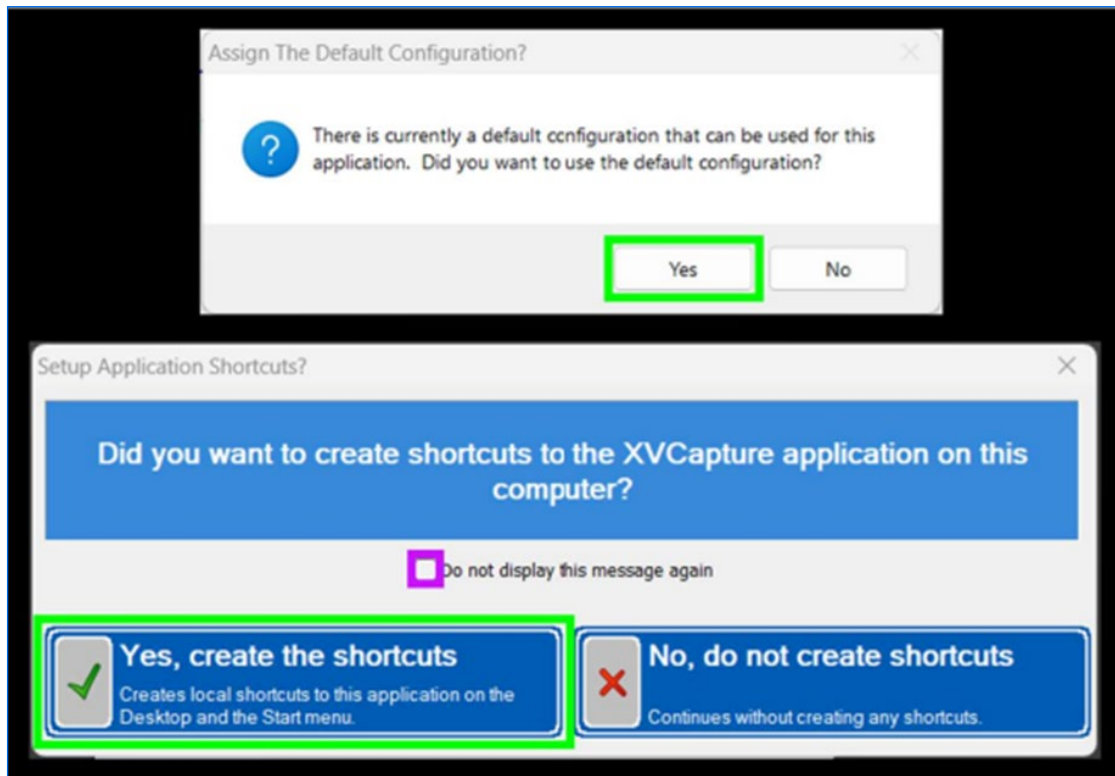


6. Open the **Extensions** folder and right click and run the **XVWebInstaller.exe** file as Administrator.

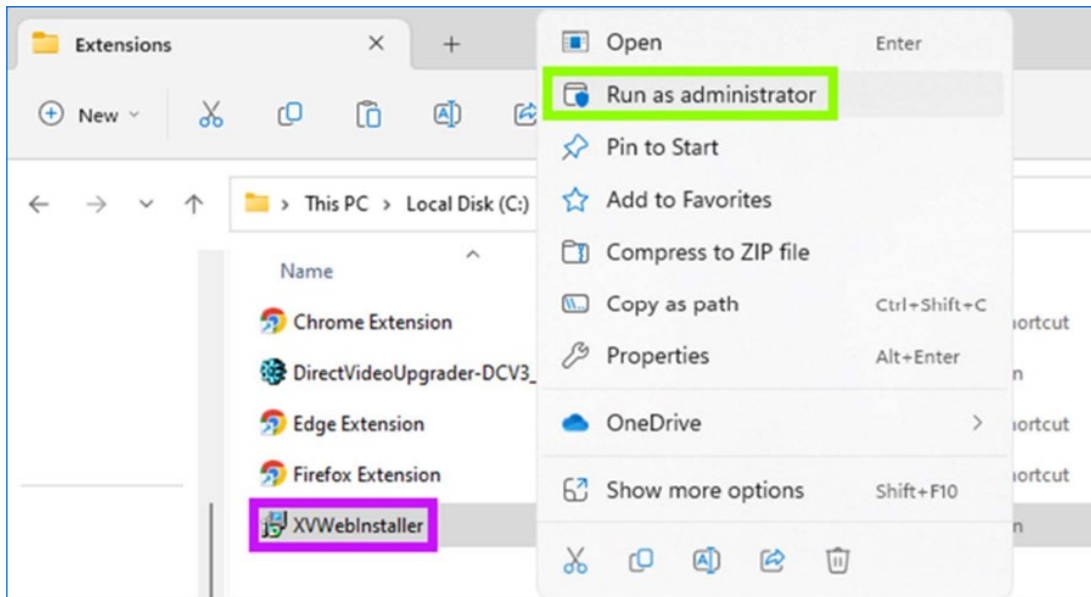

7. You will now need to install the browser extensions for any web browsers that the practice uses.

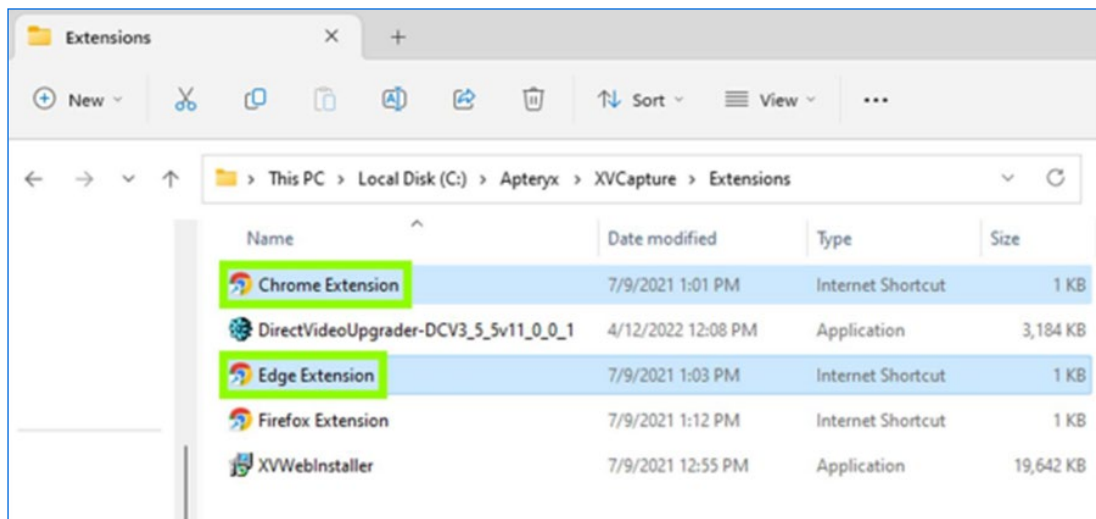

8. Right click and run any additional extension upgraders that you may need as an Administrator. Additional imaging extensions can be downloaded from this URL: <https://apteryximaging.planetdds.com/hc/en-us/articles/4412758829595-Current-Imaging-Downloads>

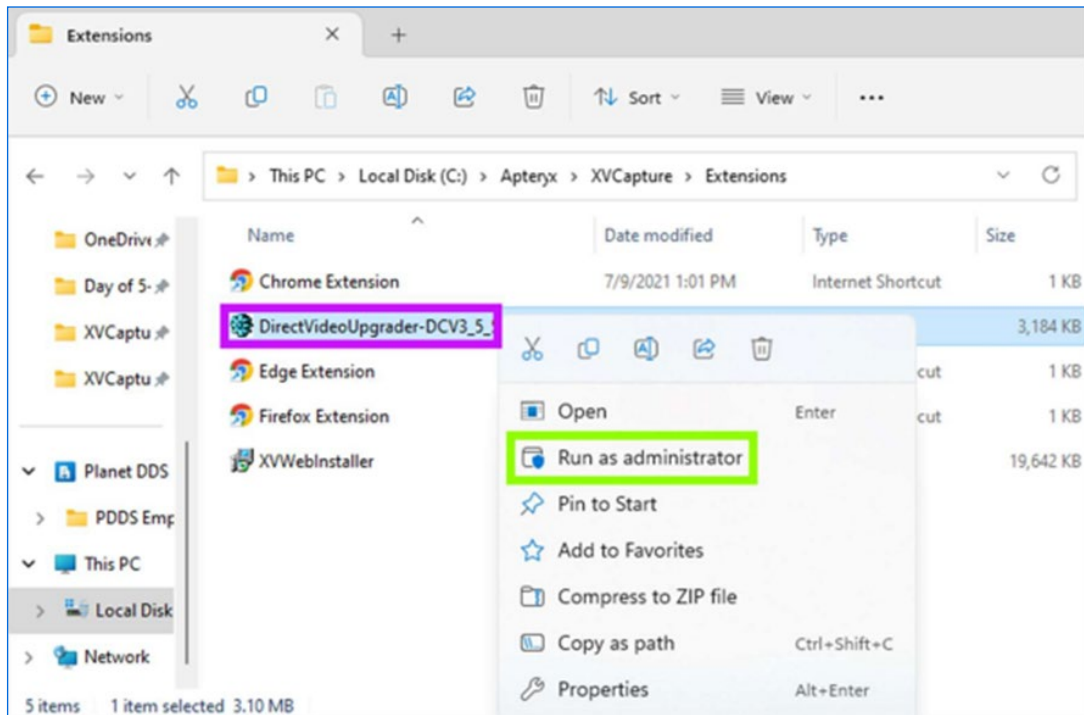



denticon | apteryx | legwork | cloud 9

Planet DDS is the leading provider of cloud-enabled dental software solutions, serving over 13,000 practices in the United States with over 118,000 users. The company delivers a complete platform of solutions for dental practices, including Denticon Practice Management, Apteryx Cloud Imaging, Cloud 9 Ortho Practice Management and Legwork Practice Marketing. Planet DDS is committed to creating value for its dental practice clients by solving the most urgent challenges facing today's dental practices nationwide.