



XVWeb Capture


INSTALLATION GUIDE

System Requirements:


- **2.0 GHz or higher processor (faster is always better, especially for servers)**
- **4GB of RAM (again, more is always better)**
- **500 GB Hard Drive**
- **CD-R/DVDR drive**
- **100/1000Mbps or higher Local Area Network (TCP/IP preferred)**

Installation

1. Navigate to your XVWeb account.
2. Once logged in, click the username in the upper right corner and select '**Capture**'.

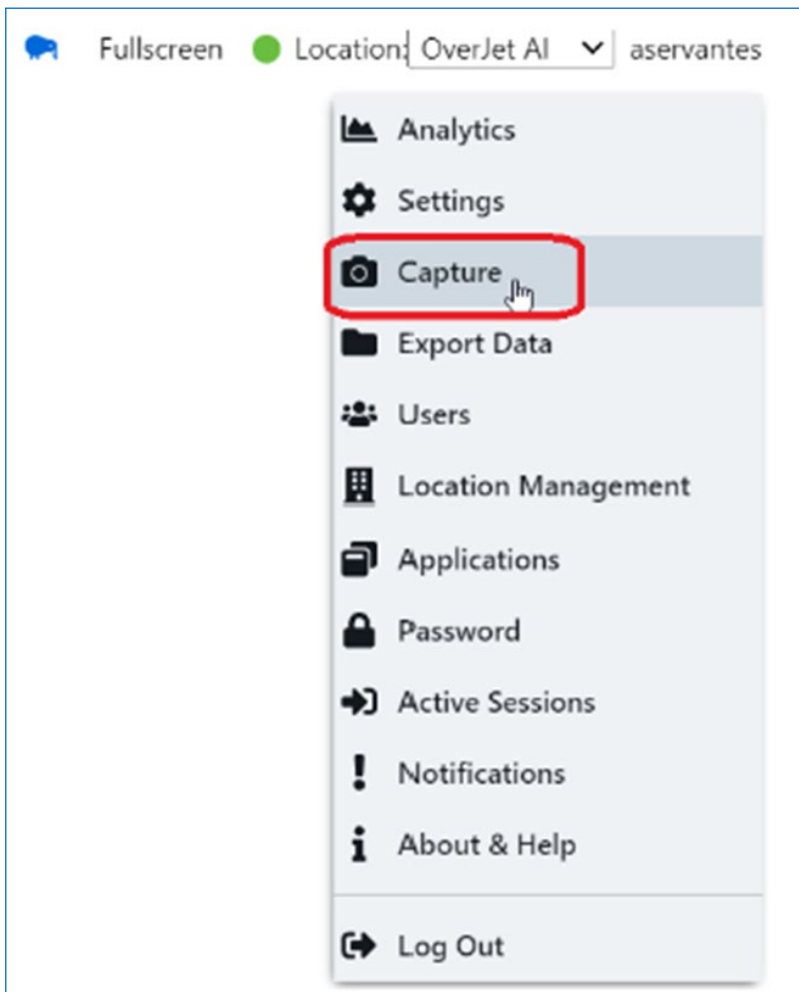


3. Click on '**Download XVWeb Capture**'.

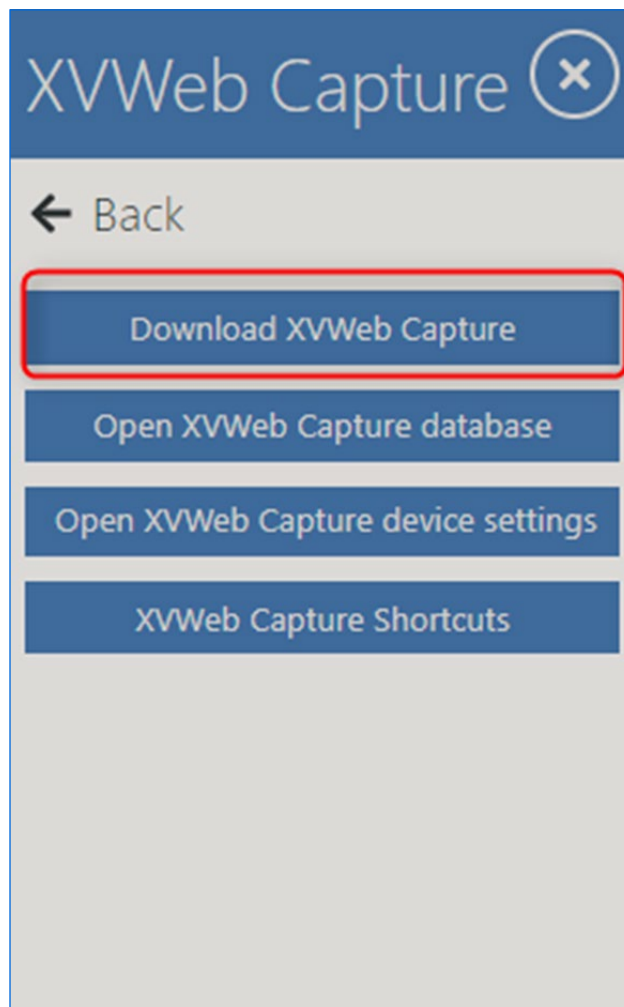

4. The download will appear in a different place depending on the browser you used. In general, it should show up in your Downloads folder on the computer no matter what browser you used.


5. You will need to locate the download titled '**XVWebCapture.exe**' and double click the file to run it.

6. Click **'Install'** on the window that pops up.

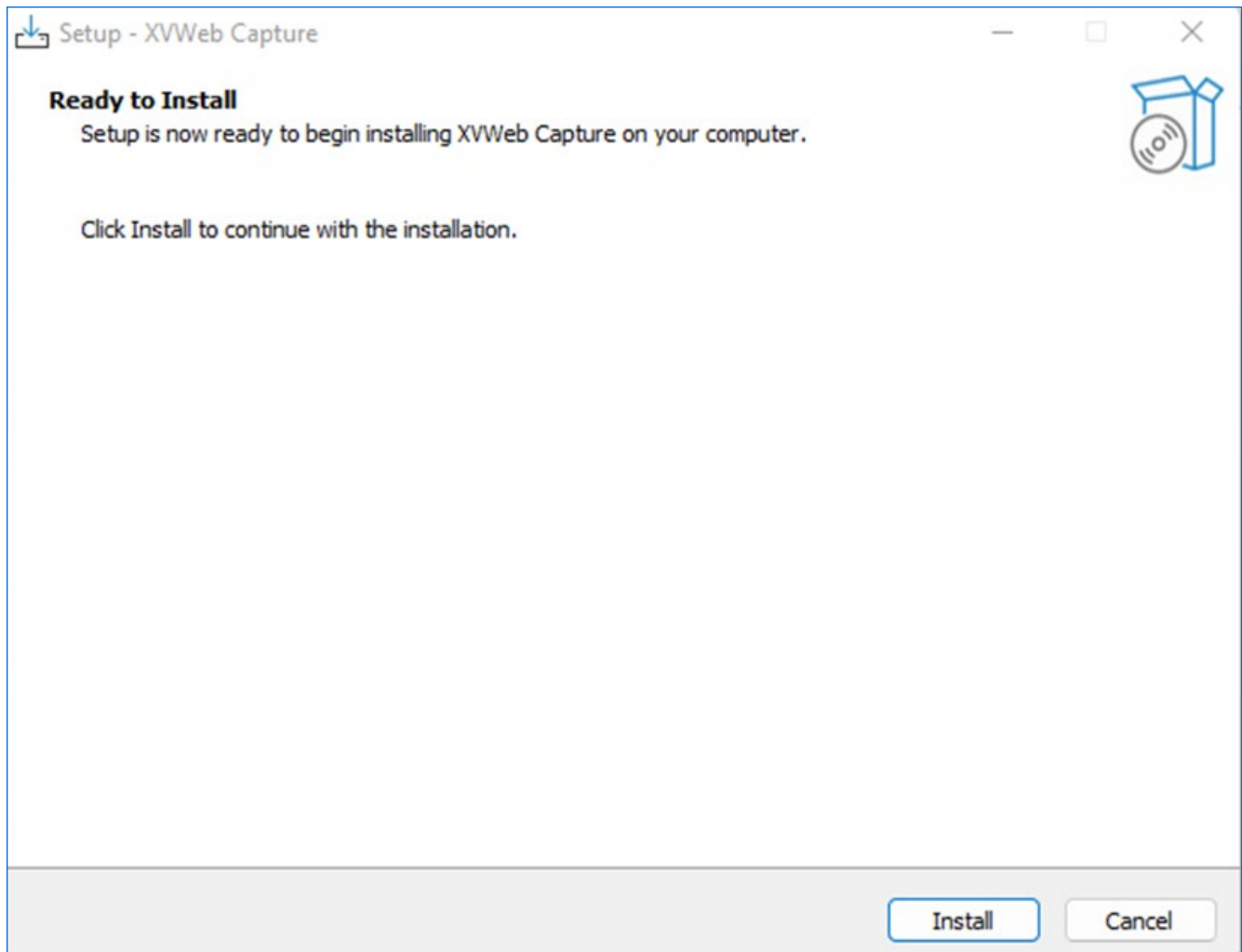


7. Depending on your computer's settings, you may see a box asking you if you would like to allow the software to modify your computer. Click **'Yes'**.

8. Select the devices you need for capturing* by clicking the device and then clicking **'Install'**.

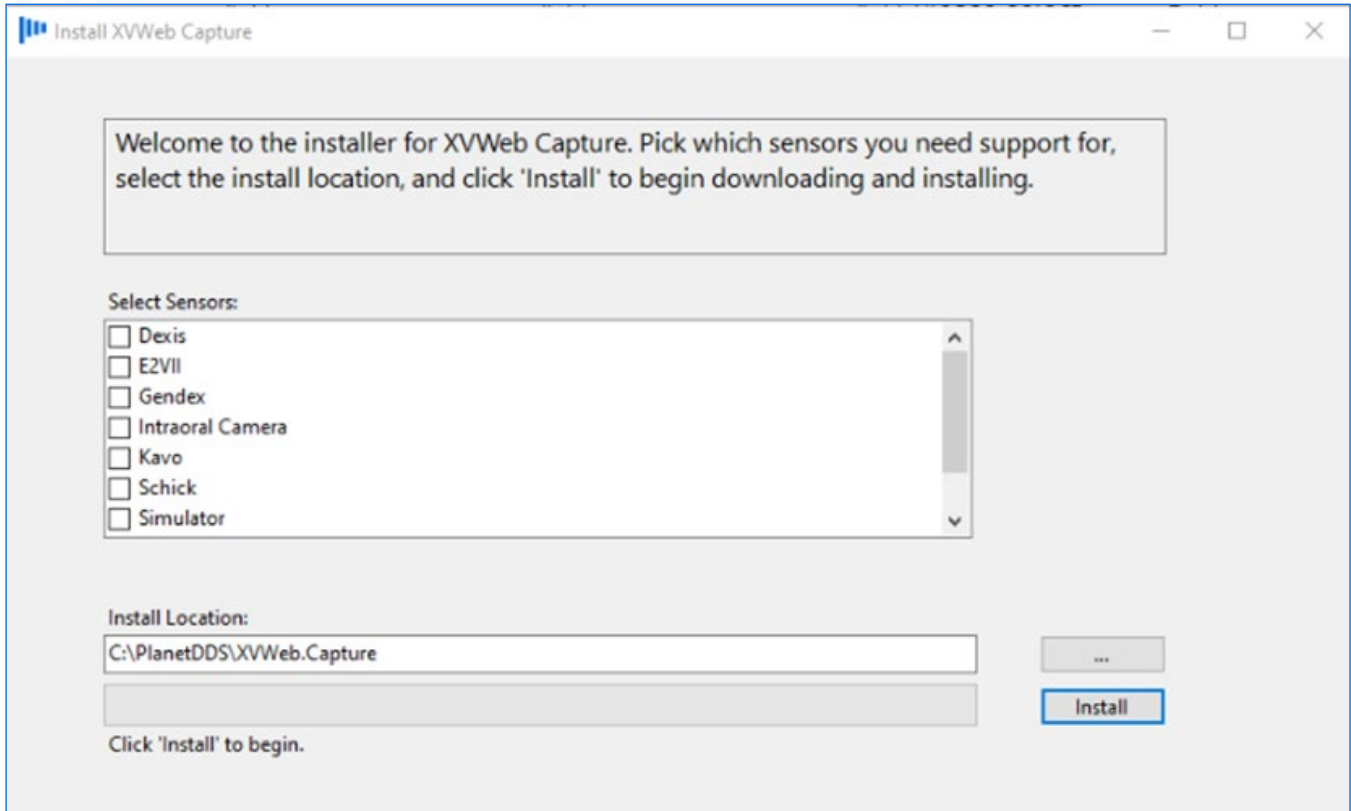

***Note:** This will not install the drivers that allow your sensor to talk to the computer; this will only install the files necessary for our web-based capture to interact with the appropriate drivers. Please see your manufacturer documentation/website for more information on correctly installing the appropriate drivers.

9. Wait for the installer to finish downloading and installing the necessary components. Once complete, a window will pop up letting you know the installation is complete. You can click **'Finish'** to close the window.

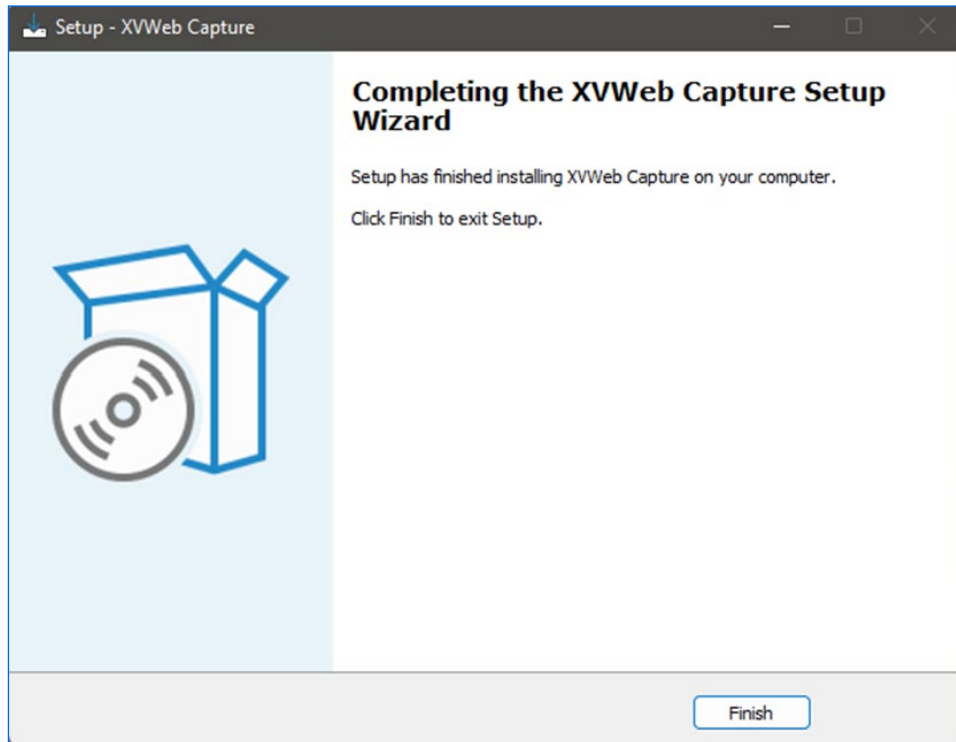

10. Navigate back to the XVWeb screen you opened in your browser in Step 1 and refresh the page (F5 on your keyboard).

11. You should now see the **'Capture'** and **'Shortcuts'** buttons appear on your XVWeb Search Page.





denticon | apteryx | legwork | cloud 9

Planet DDS is the leading provider of cloud-enabled dental software solutions, serving over 13,000 practices in the United States with over 118,000 users. The company delivers a complete platform of solutions for dental practices, including Denticon Practice Management, Apteryx Cloud Imaging, Cloud 9 Ortho Practice Management and Legwork Practice Marketing. Planet DDS is committed to creating value for its dental practice clients by solving the most urgent challenges facing today's dental practices nationwide.